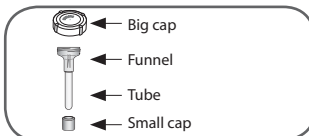
PD-PR-027 Issue 1.0
OG300 User Instructions

ENGLISH

Oragene® DNA Self-Collection Kit User Instructions

Instructions:

Do not eat, drink, smoke or chew gum for 30 minutes before giving your saliva sample.

- Spit until the amount of liquid saliva (not bubbles) reaches the level shown in picture #1.
- Hold container upright.
Close funnel with big cap.
When closed, liquid in the big cap will be released to mix with saliva.
- Hold container upright.
Unscrew funnel from tube (by twisting counter-clockwise).
- Close tube with small cap.
- Mix 5 times
Throw out funnel and big cap.

User Tips:

- Do NOT remove plastic film from the big cap.
- To make more saliva, close your mouth and wiggle your tongue or rub your cheeks.
- Some people may find it hard to spit so much saliva. It is easier to spit more if you place ¼ tsp of plain white sugar on your tongue.
- Finish spitting within 30 minutes.

Intended Use: This product is designed for the safe collection of DNA samples through human saliva.

Contents: Each collection vial contains 2ml of Oragene liquid. Before use, the solution in the cap should be clear and colorless.

Warnings: Wash with water if the Oragene liquid comes in contact with eyes or skin. Do not ingest.

Storage: Store at room temperature 15-30 °C (59-86 °F).

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
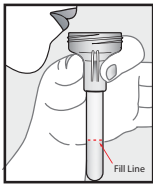
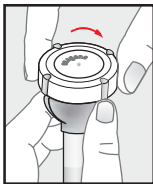
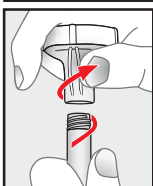
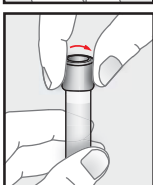
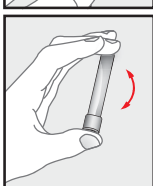
PD-PR-Q27 Issue 1.0
OG300 User Instructions

ENGLISH

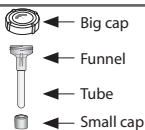Oragene® DNA Self-Collection Kit
User Instructions

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- 1 Spit until the amount of liquid saliva (not bubbles) reaches the level shown in picture #1.
- 2 Hold container upright. Close funnel with big cap. When closed, liquid in the big cap will be released to mix with saliva.
- 3 Hold container upright. Unscrew funnel from tube (by twisting counter-clockwise).
- 4 Close tube with small cap.
- 5 Mix 5 times. Throw out funnel and big cap.

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